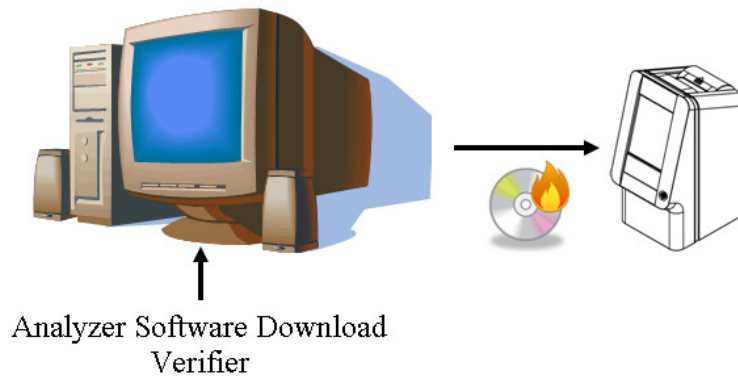


Analyzer Software Download Verifier Installation Instructions



The Installation Instructions describe how to install the Analyzer Software Download Verifier, write the Analyzer Software ISO file onto a CD and update the Blood Chemistry Analyzer software using the CD created.

Before starting you will need:

- Piccolo Xpress or VetScan VS2 Blood Chemistry Analyzer.
- Analyzer serial number.
- PC running Windows® XP, Vista or 7 OS.
- Administrator privileges on the PC.
- PC with CD writer and CD writing software.
- Blank writable CD.
- Broadband Internet connection.

Important:



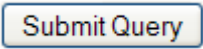
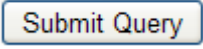
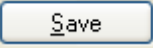
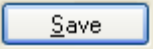
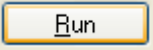
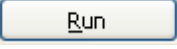
If you are upgrading to the next version of the Analyzer Software Download Verifier you must uninstall the Analyzer Software Download Verifier before installing the current version.

Uninstall the Analyzer Software Download Verifier:



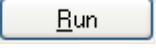
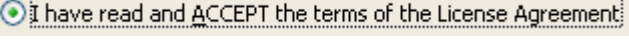
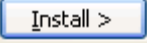
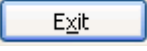


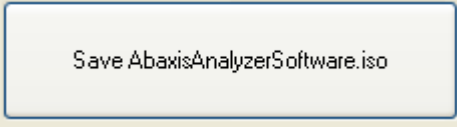
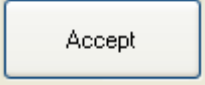
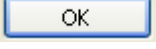
- **Windows® > Control Panel > Add or Remove Programs > Analyzer Software Download Verifier > Change/Remove > OK.**

Analyzer Software Download Verifier Installation Instructions


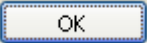
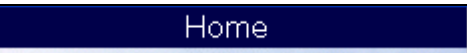





Installation Steps

Open Internet Explorer	
Navigate to http://www.abaxis.com	
Support	
Request Software	<p>GOVERNMENT SUPPORT For General Information & Request for Quotes, email GovernmentSupport@Abaxis.com</p> <p>Request Software</p>
Complete the registration form for the Analyzer Software Download Verifier.	
Submit Query An email with download information is sent to the email address you provide.	
Open the email sent by Abaxis Technical Support to the email address you provided.	
Navigate to download page using the link in the confirmation email.	http://updates.abaxis.com/?id=BC5A2B4DEB4BAF07
Copy the Registration Code from the confirmation email.	02f7c7aacf755cf6e2689ae2341174
Paste the Registration Code from the confirmation email into the download page.	<p>Registration Code</p> <input type="text"/>
Submit Query	
Select the download link Analyzer Software Download Verifier Download.	<ul style="list-style-type: none"> Analyzer Software Download Verifier Download
Save	
Save setup.exe to your desktop.	
Run	
Run setup.exe	

Analyzer Software Download Verifier Installation Instructions

<p>Accept the Abaxis End User License Agreement.</p>	
<p>Install If .NET 3.5 SP1 is already installed on the computer skip this step.</p>	
<p>Run If .NET 3.5 SP1 is already installed on the computer skip this step.</p>	
<p>Select I have read and ACCEPT the terms of the license agreement. If .NET 3.5 SP1 is already installed on the computer skip this step.</p>	
<p>Install If .NET 3.5 SP1 is already installed on the computer skip this step.</p>	
<p>Exit If .NET 3.5 SP1 is already installed on the computer skip this step.</p>	
<p>Install If .NET 3.5 SP1 is already installed on the computer skip this step.</p>	
<p>Registration Page > Enter your Email address.</p>	<p>E-mail Address software@abaxis.com</p>
<p>Registration Page > Paste the Registration Code from the confirmation email into the download page.</p>	<p>Registration Code (located in) 5c0698a9e65ec26e3b0046f3</p>
<p>Submit</p>	
<p>Select Save AbaxisAnalyzerSoftware.iso</p>	
<p>Read Release Notes > Accept Release Notes.</p>	
<p>Browse to the folder where the Analyzer Software will be saved > OK.</p>	

Analyzer Software Download Verifier Installation Instructions

This example uses Roxio Creator DE version 3.3.0. The CD writing software on your computer may be different.	
Roxio® Creator DE > Write CD	
Insert a blank writable CD into the CDR drive > Close the CD drive	
OK	
Remove the CD from the CDR drive.	
Turn ON the Abaxis Blood Chemistry Analyzer.	
Analyzer > Home	
Hand	
More Options	
CD	
Insert the Analyzer Software CD (CD drive is on the back of the analyzer) > Close drawer > (Analyzer Screen) Continue > Wait for the software update to finish.	
Analyzer > Home > Hand > Analyzer with Question Mark	
Confirm the analyzer software version has been updated.	